


HYDRO JET

OPERATING INSTRUCTIONS

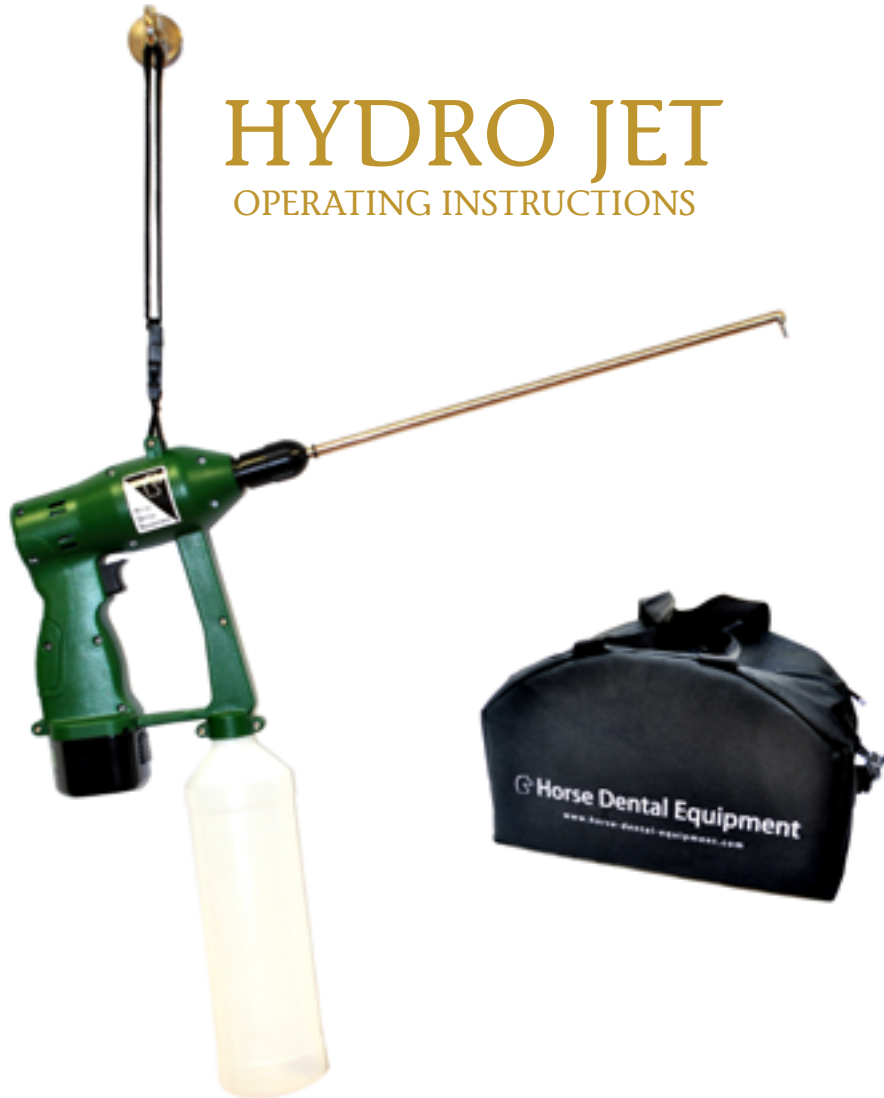

You have purchased quality professional equipment from HDE and we thank you for your trust.

Please read these operating instructions carefully, in order to optimize the use and longevity of your equipment.



USE WITH THE 1-LITER BOTTLE

Insert the tube on the nozzle at the base of the Hydro Jet, making sure that there is a strainer at the other end of the tube (PICTURE 1).

Fill the bottle with clean water and screw the bottle on the base of the Hydro Jet (PICTURE 2).

Place the desired nozzle on the Hydro Jet by fastening the plastic ring without over tightening (PICTURE 6).

Place the battery at its intended location, on the base of the Hydro Jet (PICTURE 3).

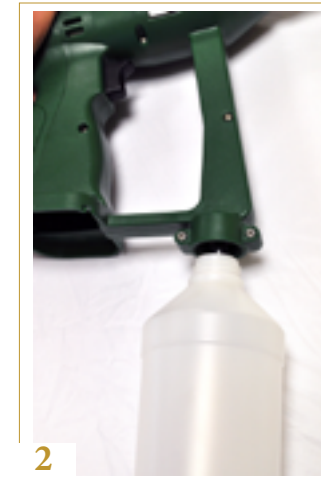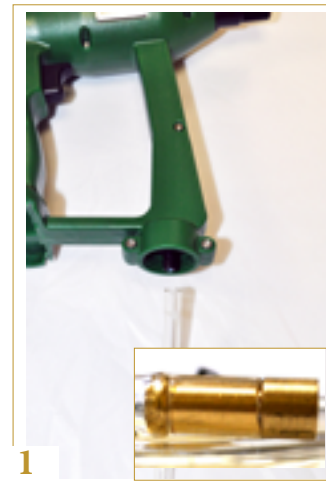


Press the trigger button to prime the pump.
The pressure on the trigger button controls the water pressure at the nozzle outlet.

WARNING : Never operate the Hydro Jet without water.


ACCESSORIES



3 DIFFERENT NOZZLES

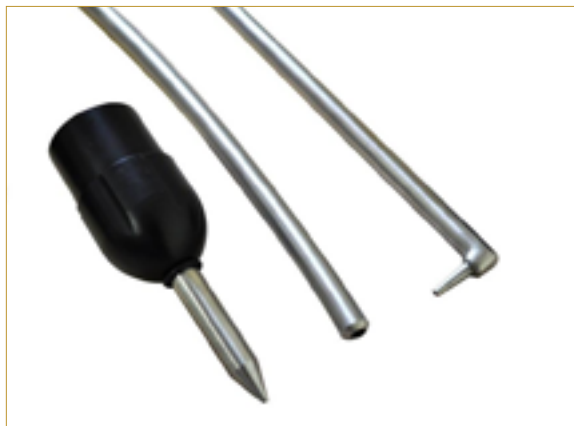

STORAGE BAG

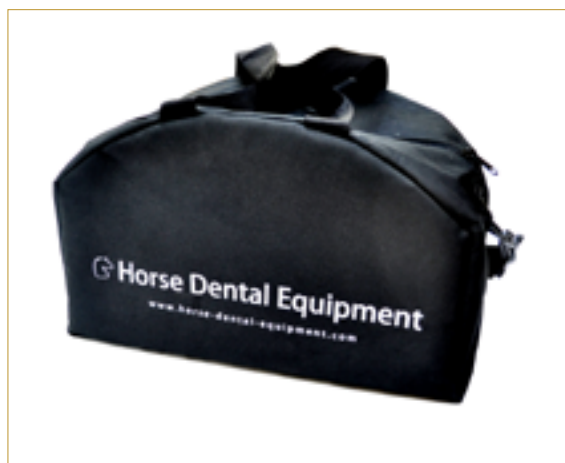

CLEANING RECOMMENDATIONS



CLEANING RECOMMENDATIONS

At the end of the working day, clean the Hydro Jet (without battery) with a dampened sponge.

Regularly clean the strainers with water and a small nylon brush.

Thoroughly dry the different elements with a clean cloth and store them in the dedicated storage bag.

WARNING : Do not clean under running water nor immerse the Hydro Jet, charger and battery.

DEFECTS

NO PRIMING OR WEAK WATER SPRAY :

- Make sure the end of the tube is well under water and that there is no air intake
- Check the battery charge
- Check the strainer
- Check the nozzles

FOR ANY PROBLEM, CONTACT YOUR LOCAL TECHNICAL CENTER

USE



WITH A BUCKET

Thanks to the 6-meter tube supplied (PICTURE 4), you can increase your water supply, by using an HDE bucket (ref. ACC_S12L) which perfectly corresponds to this use (PICTURE 5).

Insert the tube on the nozzle at the base of the Hydro Jet and make sure that there is a strainer at the other end of the tube (PICTURE 1).

Immerse the tube in the bucket filled with clean water.

Make sure the tip of the tube remains submerged underwater, at the bottom of the bucket, throughout the working phase.

Place the desired nozzle on the Hydro Jet by fastening the plastic ring without over-tightening (PICTURE 6).



Press the trigger button to prime the pump.
The pressure on the trigger button controls the water pressure at the nozzle outlet.

WARNING : Never operate the Hydro Jet without water.

USE



CHANGING THE SPRAY ANGLES

If needed, you can easily change the spray angles of the nozzles (diastema/rinsing).

Slightly unscrew the plastic ring and direct the tip manually (PICTURE 7).



PRACTICAL

Between two uses, you can easily suspend the Hydro Jet, using the strap supplied.

The HDE magnetic hook (ref. ACC_CA) is particularly convenient and can easily attach to any metallic part in the stalls.

BATTERY



RECHARGING THE BATTERY

Place the battery on the charger and connect it to the electric outlet.
When the charging indicator is on green, the battery is charged and ready to use.

BATTERY LIFE

The battery life is approximately 1 hour (continuous use).



BATTERY



CHARGER

HORSE DENTAL EQUIPMENT
HDE North America LLC
2316 Arbor St.
Houston TX 77004

Phone: +1 (713) 444-2531
contact@horse-dental-equipment.us

OFFICIAL HDE TECHNICAL CENTER
Swisher Dental Equipment
22815 Commercial Lane #A
Tomball, TX 77375

Phone: + 1 (281)351-8660
contact@horse-dental-equipment.us

US MASTER DISTRIBUTOR

Jorgensen
Inc. 1450 Van Buren Ave
Loveland, CO 80538

Office: 1-800-525-5614 x 202
Contact: Jill Pickard
jill@jorvet.com
www.jorvet.com