

# CLEANING RECOMMENDATIONS

You have purchased quality professional equipment.  
Please find hereafter advice on how to optimize its use and longevity.

## GENERAL RECOMMENDATIONS

- Make sure each aspect of your equipment is always clean & dry before storage
- HDE recommends at least 1 annual checkup (or every 500 horses) of your complete equipment at an official HDE technical center\*
- HDE does not cover damages caused by the use of bleach
- Find all of our recommendations in our products' instruction manuals

<http://www.horse-dental-equipment.us/downloads/>



## DIAMOND BURRS

1. Start the engine with the handpiece attached
2. Run it at high speed with the head pointing down to remove any remaining water
3. Clean the diamond burr by running it over the HDE stainless steel brush
4. Continue until the remains of dentine have completely disappeared
5. Clean with a cloth moistened with clean water and a disinfectant
6. Thoroughly dry with a clean, soft cloth
7. Store the burr in its dedicated place within your clean, dry case

**The burr is resistant to water projection, do not fully immerse in water.  
Always use clean water without corrosive nor abrasive products.  
Only use the HDE stainless steel brush.**

## DRIVE SHAFT

1. Clean the outside sheath with a damp sponge
2. Dry thoroughly with a clean, soft cloth
3. When finished working, detach the drive shaft from the engine and the handpiece
4. Store the drive shaft in your clean, dry case

**Make sure to not overbend the driveshaft.**



## ENGINE

1. Turn off the engine before cleaning
2. Clean the engine with a damp sponge
3. Thoroughly dry the engine with a clean, soft cloth
4. Store the engine in your clean, dry case

**HDE recommends to use the engine at medium speed (#3).**

## CASE

1. Clean the case with a damp sponge
2. Thoroughly dry the case with a clean, soft cloth
3. Store in a clean and well ventilated place

**Make sure the case is opened from time to time, to ensure your equipment is well ventilated.**

